



Transitional device helps get patients out of the pain cycle

QuickSplint™ is a transitional discluding device intended to be worn by the patient overnight. QuickSplint™ reduces posterior bite force, thus fostering an immediate and pronounced relaxation in the masticatory muscles. QuickSplint™ is simple to fabricate - *taking only minutes* - and completely comfortable for the patient to wear. Best yet, in addition to being a short-term TMD solution, QuickSplint™ may possess a variety of other applications in your dental practice.

Specialists are incorporating QuickSplint™ as a new treatment protocol for Endodontic, Periodontic, Implant and Cosmetic Dentistry procedures, to decrease post-operative complications, protect restorations, and address occlusal trauma at the time of surgery.

Generalists are using QuickSplint™ as a diagnostic tool for patients in acute pain, and for patients who have not previously been receptive to bruxism treatment. QuickSplint™ allows immediate placement at the time of consult when the patient is symptomatic and perhaps most receptive to treatment. Following treatment with QuickSplint,™ the patient's immediate pain relief is enough to affirm to them the efficacy of this novel dental device and in intraoral appliance therapy overall. QuickSplint™ is reimbursable and designed for temporary use – to be followed by a definitive solution of your choice, including custom-fabricated full-arch final appliances or referral to a specialist.

How it works: QuickSplint™ initiates occlusal appliance therapy, addressing parafunctional behavior by preventing contact between the posterior teeth. This removes or interrupts occlusal memory, allows for condylar seating, interferes with habitual bite force, allows free movement of the mandible, unlocks any interference and allows for muscle recovery.

For patients in pain and patients who brux, a 2 to 4 week treatment period is usually sufficient to determine whether oral appliance therapy will work for them. In many cases the patient can see grind marks on the underside of QuickSplint™ from bruxing. This waiting period may coordinate with the time required for insurance authorizations and lab fabrication of a suitable full-arch appliance. For use following surgery, for post-operative healing and protection of new restorations, QuickSplint™ can be worn for several months.

QuickSplint™ has two components: A) a rigid plastic shell that is filled with bite registration material to customize each appliance to supporting dictation and can be fit for both upper or lower arch and B) an insert area to be filled with suitable fast-set impression material. QuickSplint™ becomes completely individualized when the patient is instructed to bite down to level the bite registration and hold for a few short minutes. Excess material is trimmed and QuickSplint™ is delivered to the patient in the convenient storage container provided.

QuickSplint™ comes in a carton of 6 units for \$150 (\$25 per unit) and is reimbursable under D8210 (defined as Removable Appliance Therapy, minor treatment to control harmful habits. QuickSplint™ is FDA approved, patented, and made in the USA.



Order online at www.QuickSplint.com or call 800-760-0526