




Titrating the EMA appliance

The **EMA** oral appliance from Myerson increases airway space by advancing the mandible using interchangeable elastic straps. EMA elastic straps come in **9 different lengths** (see figure A), and **4 different strengths** (see figure B). The shorter the ema elastic strap, the farther the mandible is advanced (see figure C). **36 different straps** allow for the most effective repositioning with maximum comfort.


9 different lengths allow for 1 mm advancement of the mandible

Figure A

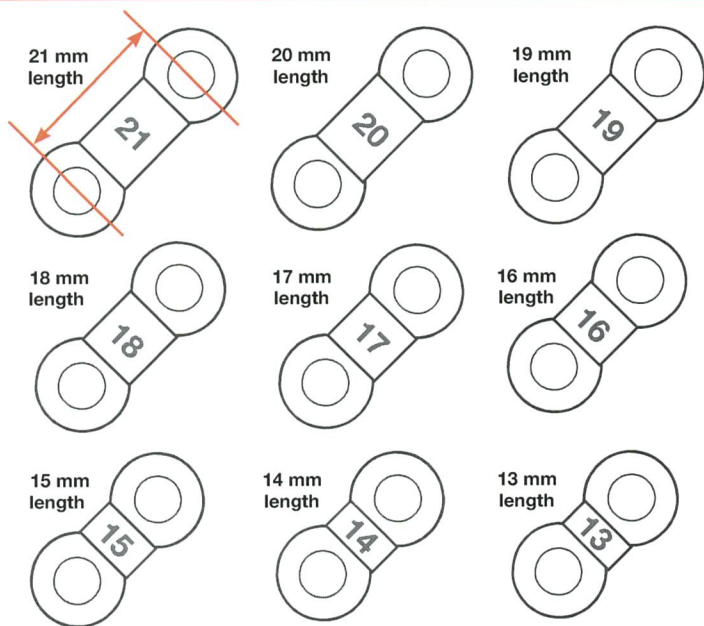

Each EMA elastic strap comes in 4 different strengths, indicated by color.

Figure B

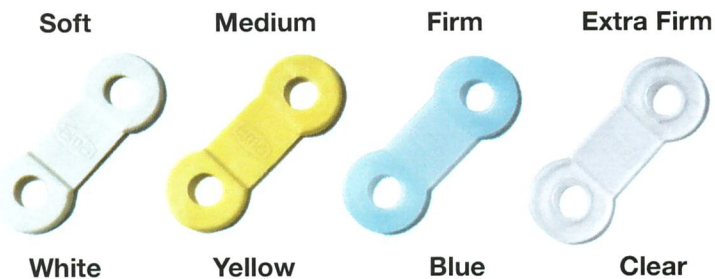

EMA elastic straps are 100% latex free.



MANUFACTURED BY
myerson LLC
 5106 north ravenswood
 chicago il 60640-2713
 usa
 phone 800 423 2683

EUROPEAN REPRESENTATIVE
myerson limited
 34 railway approach
 station road
 harrow middx, HA3 5AA
 united kingdom
 phone +44 0 20 8863 9044




EMA® is a trademark of frantz design inc.
 US Patent Numbers: 6,109,265; D611,153 & Patent Pending

The shorter the EMA elastic strap, the farther the mandible is advanced.

Figure C

EMA buttons are 27 mm apart*

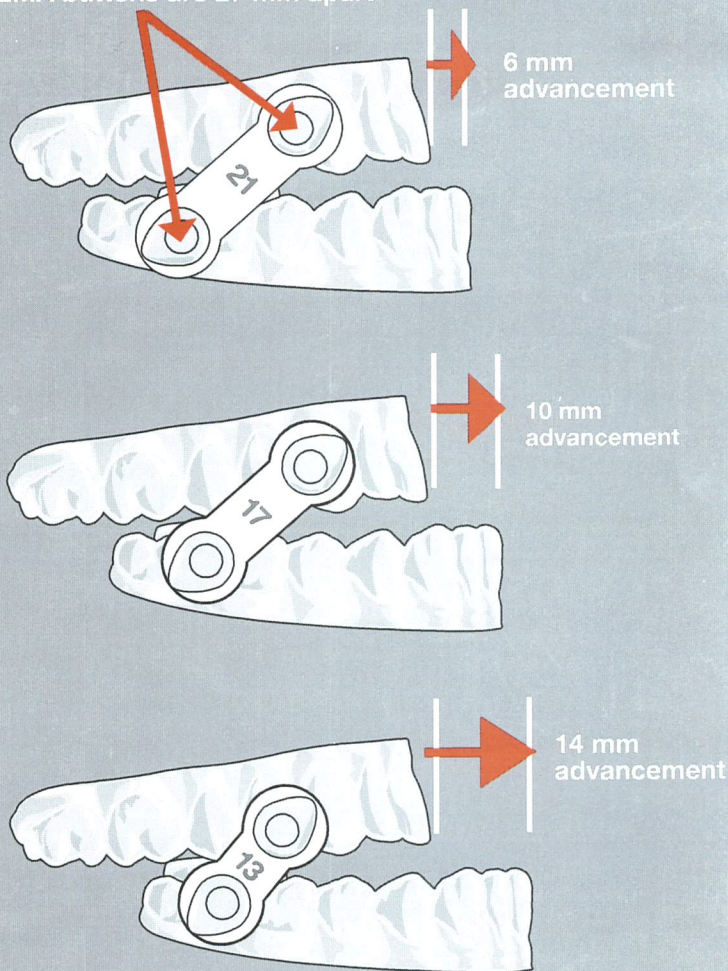

EMA® STRAP LENGTH	DISTANCE BETWEEN EMA BUTTONS	
	27 mm	25 mm
21 mm	6 mm	4 mm
20 mm	7 mm	5 mm
19 mm	8 mm	6 mm
18 mm	9 mm	7 mm
17 mm	10 mm	8 mm
16 mm	11 mm	9 mm
15 mm	12 mm	10 mm
14 mm	13 mm	
13 mm	14 mm	

*Myerson recommends a distance between the buttons of 27 mm in the centric position. If the patient's range of mandibular advancement is limited to 9 mm or fewer, a distance of 25 mm distance is recommended.